

APPLICATION OF THE MASK




This is what the mask should look like when you're wearing it. The colorful side of the mask, which will be either blue, green, yellow, or red, faces outwards, and the wire nose bridge should be pinched at the top of the nose.

Polypropylene material which helps block droplets from getting in, or out.

FRONT



BACK

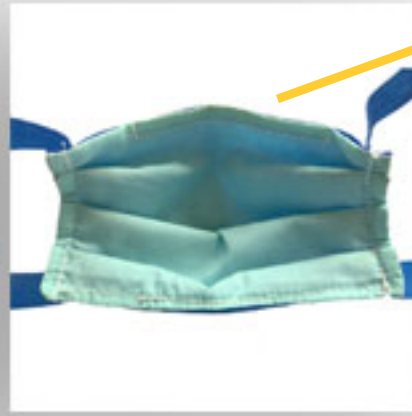

Polycotton material that is lightweight, breathable and soft against your face.



You can either tie the top straps together, and the bottom straps together, OR use the plastic cord locks included. To put the plastic cord locks on, fold over the ends of the top two straps and feed them into the cord lock opening.

CLEANING & CARE INSTRUCTIONS

Hand-washing:

We recommend hand washing the masks in warm water using dish soap – Dawn works great, but whatever dish soap you have is fine. Wash thoroughly, rinse, ring out and hang dry.



Machine Washing:

You can also put the masks through the washer and dryer however be sure to use gentle cycle. If you put them in the washer/dryer, we recommend removing the plastic cord locks.



EQUIP
YOUR
SPACE

Equip, Inc.

Support@equipinc.com

www.equipinc.com

WARNING: CLEAN BEFORE EACH USE

Do NOT put masks in the Microwave.

Do NOT put masks in an Autoclave.